



## LC-01

Sturdy's LC-01 Handpiece Cleaner cleans and lubricates handpiece instruments in a single convenient cycle.

It has the ability to automatically maintain up to four handpieces at a time and adaptable different brands of high and low speed handpieces. This design meets the requirements of modern dental practices, which are busy and do not want to compromise on the quality of maintenance.



Reliable Sterilization Partner

# Handpiece Cleaner

## LC-01

### Features:

1. Fast & efficient maintenance:

Adaptable to different types of high and low speed handpieces; attached directly.




2. The automatic rotation function during the cycle guarantees each handpiece is cleaned and lubricated thoroughly.

6. Press the "AIR" button; activation of air compressor to flush out all residual solution.



7. Filter sponges absorb residual spray mist. The sponges are easy to be replaced.

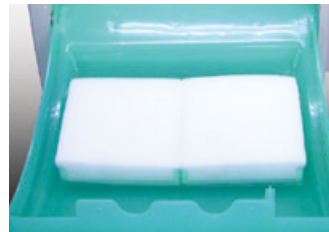

8. The front cover can be completely removed by the touch of a button for easy cleaning.

9. The oil level indicator helps users to check the volume of the oil conveniently.



3. Simple and easy-to-understand control panel:

- Cycle "S" (approx. 35 seconds):  
Short cycle for high speed handpieces.
- Cycle "M" (approx. 45 seconds):  
Medium cycle for low speed handpieces.
- Cycle "L" (approx. 50 seconds):  
Long cycle for all hand pieces.

4. Error indicator:

The indicator light illuminates when the door isn't closed completely; the machine is not operational.



5. Complete alarm:

A buzzer sounds when the cycle is complete.

### SPECIFICATION

Available Model	LC-01 Rotation port × 1 pc (Low Speed) Standard port × 3 pc (High Speed)
	LC-01A Rotation port × 2 pc (Low Speed) Standard port × 2 pc (High Speed)
	LC-01B Rotation port × 3 pc (Low Speed) Standard port × 1 pc (High Speed)
Power Supply	110V - 120V, 50/60 Hz, 0.5 A 220V - 240V, 50/60 Hz, 0.3 A
Air Pressure	0.35 - 0.6 Mpa (3.5 - 6 kgf/cm <sup>2</sup> )
Operation Condition	5 - 40°C (41 - 104 °F)
Oil Tank Capacity	300ml (Max. level)
Dimensions	360(W) × 290(D) × 340(H) mm
Weight	Approx. 7.3 kg
Standard Accessories	Power cord x 1 Maintenance oil x 1 (1 liter bottle) Oil replenishment funnel x 1 Fuse x 2 Air tube x 1 Y-type quick-connecting joint x 1 Filter sponge x 6 (Pack of 2) Magnet x 1 Test bur x 2 O-ring x 2

**Sturdy**  
Welllell Group

No. 168, Sec. 1, Zhongxing Road, Wugu District,  
New Taipei City 24872, Taiwan  
Tel: +886-2-2981-5245

Fax: +886-2-2984-8208(Office)  
Email: [foreign@sturdy.com.tw](mailto:foreign@sturdy.com.tw)

[www.sturdy.com.tw](http://www.sturdy.com.tw)



Sturdy reserves the rights to improve the equipment shown without prior notice.

