

Autoclave

SAT Series

SAT are digital temperature control autoclaves with pre & dry vacuum. Designed for Hospital medical, laboratory, biological science, pharmacy industry, reliability testing equipment.



Autoclave

SAT Series

Features

SAT Series

- Fractioned pre vacuum air removal
- Independent steam generator
- Built-in temperature recorder
- Cycle progress indicator light
- Double locking safety system
- Special liquid program
- Stand by function
- Cylindrical chamber volume: 100, 143, 196, 350 (L)
- Square Chamber volume: 110, 260, 454, 680, 848 (L)



Autoclave

SAT Series

SAT Series External View : Loading door

SAT

A Recorder



B Door handle



C It can change into square door.



Reliable Sterilization Partner

Autoclave

SAT Series


SAT Series External View : Unloading door

SAT SERIES WITH DOUBLE DOOR- SAT-400D/450D /500D /600D
SAT-S0110D/S0260D/S0454D/S0680D/S0848D

A Door handle




B It can change into square door.

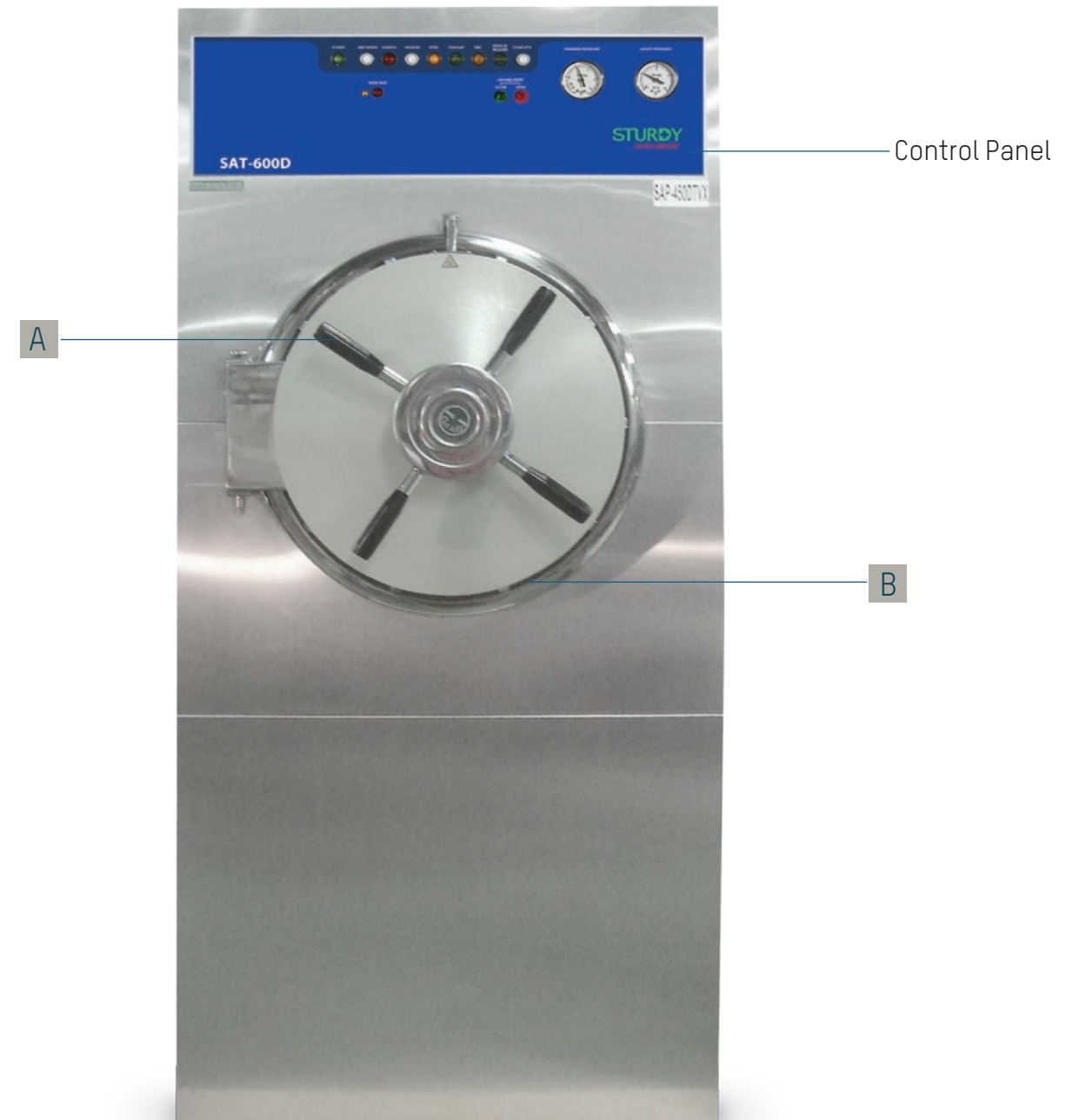

Autoclave

SAT Series

SAT Series Control Panel: Loading door

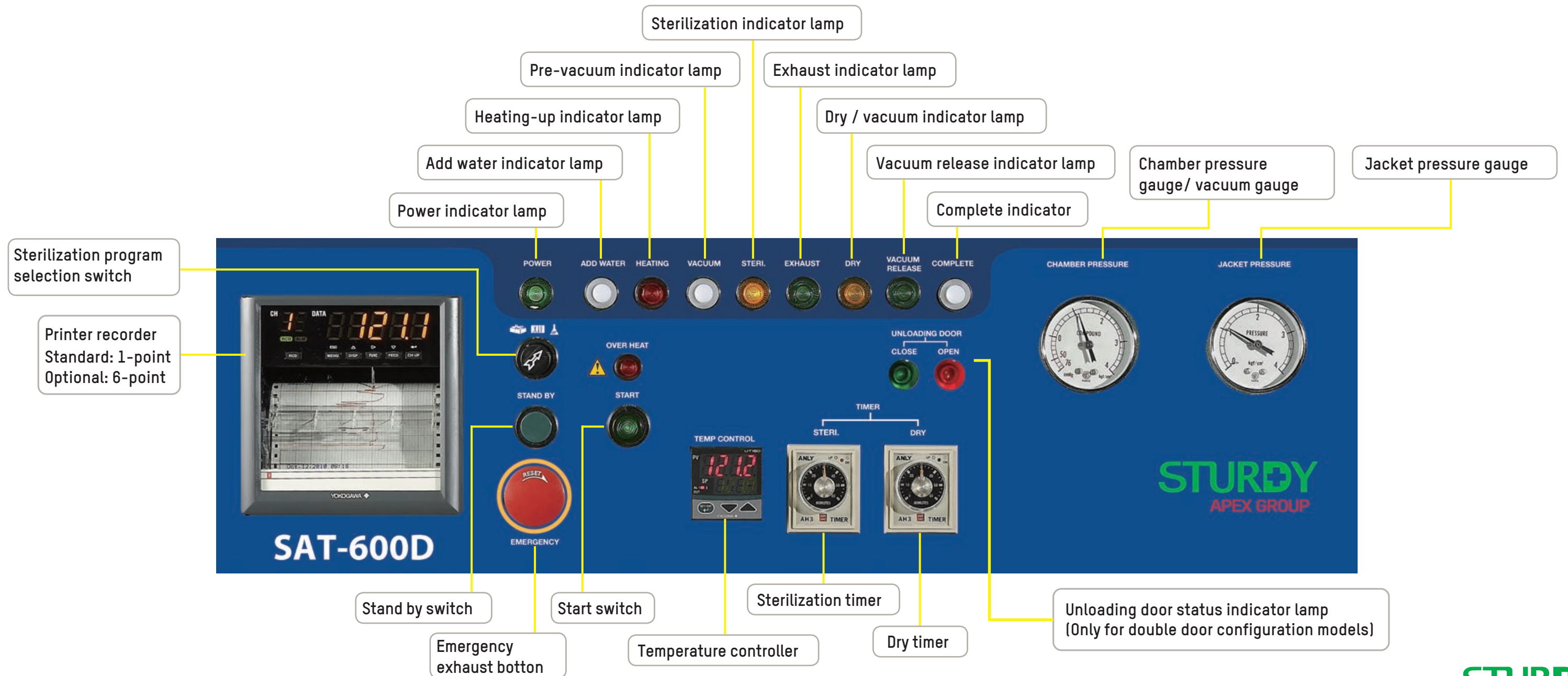


Reliable Sterilization Partner



Autoclave

SAT Series

SAT Series Control Panel : Unloading door

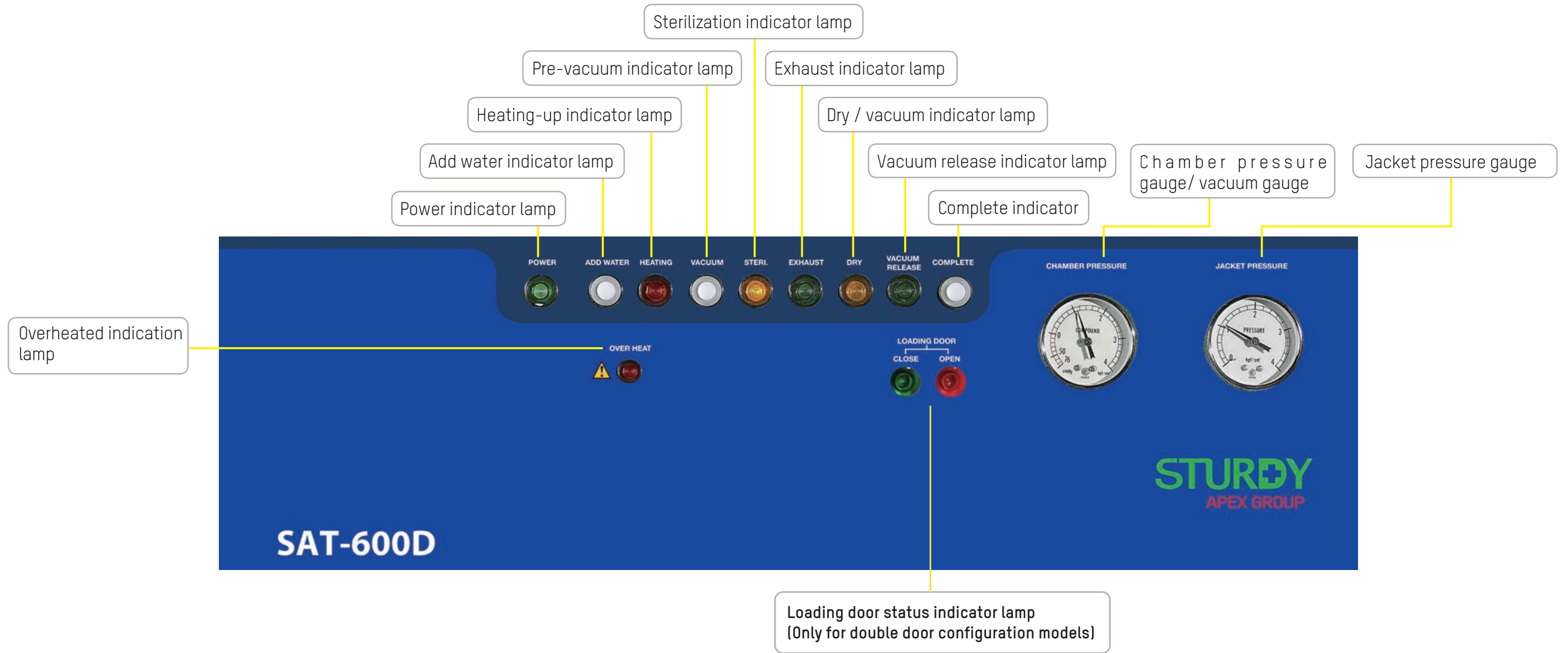

Autoclave

SAT Series

Certifications

- ISO 13485:2016 Quality Management System - Medical Devices
- PED 2014/68/EU Pressure Equipment Directive

SAT Series

SAT Series Specifications

Autoclave

SPEC/Model NO.	SAT-400	SAT-450	SAT-500	SAT-600	SAT-S0110	SAT-S0260	SAT-S0454	SAT-S0680	SAT-S0848
Chamber Capacity (L)	100	143	196	350	110	260	454	680	848
Chamber Size									
Diam x Depth (mm)	400x800	450x900	500x1000	610x1200	410x410x660	510x510x1000	610x610x1220	610x915x1220	610x915x1520
Overall Dimensions									
W x H x D (mm)	800x1800x1150	800x1800x1250	800x1760x1440	900x1800x1550	950x1800x1350	950x1800x1550	1450x1925x1800	1740x2060x1960	1740x2060x2260
Voltage (V)	230/380(Tri phase)								
Heater (KW)	7		9	12	9	12	15 x 2		16 x 2
Water Fill *	Piped								
Chamber Type	Cylindrical				Square				
Temperature Options (°C)	105-136								
Selectable Sterilization Timer	0-60 min.								
Selectable Dry Timer	0-60 min.								
Digital Printer	N/A								
6-Point Temp Sensor Recorder	0								
1-Point Temp Sensor Recorder	S								
Liquid	S								

* "Piped" - uses local water plumbing with auto-fill.

Autoclave

SAT Series

SAT Series (With Double Door) Specifications

SPEC/Model NO.	SAT-400D	SAT-450D	SAT-500D	SAT-600D	SAT-S0110D	SAT-S0260D	SAT-S0454D	SAT-S0680D	SAT-S0848D	
Chamber Capacity (L)	100	143	196	350	110	260	454	680	848	
Chamber Size										
Diam x Depth (mm)	400x800	450x900	500x1000	610x1200	410x410x660	510x510x1000	610x610x1220	610x915x1220	610x915x1520	
Overall Dimensions										
W x H x D (mm)	800x1700x1300	800x1700x1400	800x1700x1450	900x1800x1640	850x1710x1110	1025x1795x1480	1170x1850x1800	1740x2073x1810	1740x2073x2130	
Voltage (V)	230/380(Tri phase)									
Heater (KW)	7		9	12	9	12	15 x 2		16 x 2	
Water Fill *					Piped					
Chamber Type	Cylindrical				Square					
Temperature Options (°C)	105-136									
Selectable Sterilization Timer	0-60 min.									
Selectable Dry Timer	0-60 min.									
Digital Printer	N/A									
6-Point Temp Sensor Recorder	0									
1-Point Temp Sensor Recorder	S									
Liquid	S									

* "Piped" - uses local water plumbing with auto-fill.

Autoclave

SAT Series

SAT Series Specifications

SPEC/Model NO.	SAT-400	SAT-450	SAT-500	SAT-600	SAT-S0110	SAT-S0260	SAT-S0454	SAT-S0680	SAT-S0848
Temperature Indicator					LED display				
Pressure Indicator * ²					Gauge				
Progress Indicator (bar or lights)					S				
Power Indicator					S				
Pressure Auto Lock					S				
Reset					S				
Standby * ³					S				
Program Selector					S				
Pre-Vacuum					S				
Post Dry Vacuum					S				
Water Pump					0				

Note: "S" - Standard configuration. "0" - Optional configuration.

*² "Gauge" - Includes both Jacket and Chamber pressure gauges.

*³ "Standby" - Allows the Big Autoclave to maintain jacket temperatures between sterilization sessions.

Other notes:

- Standard Chamber / Jacket / Door / Housing are made of Stainless Steel#304. #316 Stainless steel optionally available for Chamber / Jacket / Door.
- Heater material is Stainless Steel #316.
- Standard Accessories: Tray*1, Glass Viewer Tube * 2
- Optional Accessories: Trolley / Plate Set / Basket
- Sturdy also offers customized sizes up to 2000 L.

Reliable Sterilization Partner

SAT Series

Autoclave

SAT Series (With Double Door) Specifications

SPEC/Model NO.	SAT-400D	SAT-450D	SAT-500D	SAT-600D	SAT-S0110D	SAT-S0260D	SAT-S0454D	SAT-S0680D	SAT-S0848D
Temperature Indicator					LED display				
Pressure Indicator * ²					Gauge				
Progress Indicator (bar or lights)					S				
Power Indicator					S				
Pressure Auto Lock					S				
Reset					S				
Standby * ³					S				
Program Selector					S				
Pre-Vacuum					S				
Post Dry Vacuum					S				
Water Pump					0				

Note: "S" - Standard configuration. "0" - Optional configuration.

*² "Gauge" - Includes both Jacket and Chamber pressure gauges.

*³ "Standby" - Allows the Big Autoclave to maintain jacket temperatures between sterilization sessions.

Other notes:

- Standard Chamber / Jacket / Door / Housing are made of Stainless Steel#304. #316 Stainless steel optionally available for Chamber / Jacket / Door.
- Heater material is Stainless Steel #316.
- Standard Accessories: Tray*1, Glass Viewer Tube * 2
- Optional Accessories: Trolley / Plate Set / Basket
- Sturdy also offers customized sizes up to 2000 L.

SAT Series

Autoclave

Comparison Table - Floor Stand Series Feature Comparison

Model NO.	HP	SAT	SAP	Pointer-Auto	Pointer-Manual
Specification					
Control System	Digital Temp. PLC Control		Mechanical Pressure Control Optional: (T) for Digital Temp. Control	Mechanical Pressure Control	
Program for selection Wrapped/Unwrapped/Liquid	Yes			N/A	
Air Removal	Vacuum Pump		Gravity		
Pre-Vacuum Function	Yes		N/A		
Post(Dry)-Vacuum Function	Yes		Optional (V)		N/A
Dry function	Yes (Auto)				Yes (Manual)
Printer & USB storage	Yes	N/A			
1-point temp. recorder	N/A	Yes	Optional (X)	N/A	
6-point temp. recorder	N/A	Optional (Y)		N/A	
Double Door	Optional (D)			N/A	
Fill water system* (Water pressure 2 bar required)	Auto			Manual Optional (B) for auto	Manual
Chamber/Jacket Pressure display	Gauge / LCD Display		Gauge		
Stand-by Switch	Yes		N/A		
Emergency/Reset Switch	Yes			N/A	
Add-Water Pump	Optional (W)			Optional (BW)	N/A
Water level Protection	Yes			Optional (L)	
Pressure door auto-lock	Yes		Optional (K)		

Option Codes (e.g. Optional (V)) listed in the table above are only reference codes used by other documentation.

Autoclave

SAT Series

Optional Codes

Option Codes (e.g. Optional (V)) listed in the table above are only reference codes used by other documentation.

D: DOUBLE DOOR DEVICE

W: ADD WATER PUMP

Y: 6-POINT TEMP. RECORDER

Autoclave

SAT Series

Accessories

A Basket



B Plate Set



C.1 Optional Trolley



D Filter



C.2 Optional Trolley



Reliable Sterilization Partner

STURDY
APEX GROUP